

ENDODONTICS

Well-Prep

Creamy EDTA Root Canal Cleaner



Efficient cleaning
and easy preparation
of root canal



Optimal
flowability



Open
dentinal tubules



Encouraging
decalcification of dentin
and antibacterial effect


INDICATIONS



Removal
of debris



PHYSICAL PROPERTIES

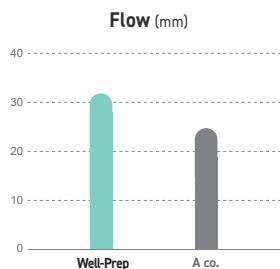

PACKAGE

ALL PRODUCTS		PRODUCT CODE	MFDS Reg. CODE
Well-Prep	6g x 3syringes, Disposable tips	WE118-K000	Well-Prep


Procedure of treatment




1. Inject a proper amount of Well-Prep into the pulp chamber.



2. File and ream canal in conventional manner.

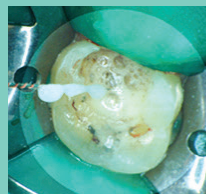

3. With NaOCl, clean the root canal.

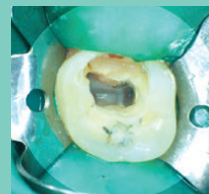

4. Re-apply Well-Prep if necessary until cleaning is complete.