Liquid Strip

[en] Instructions for Use

Water-soluble glycerine gel

[de] Gebrauchsinformation

Wasserlösliches Glyceringel

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Gel di glicerina idrosolubile

[es] Instrucciones de uso

Gel de glicerina hidrosoluble

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Gel de glicerina hidrossolúvel

[sv] Bruksanvisning

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[da] Brugsanvisning

Vandopløselig glyceringel

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Vesiliukoinen glyseroligeeli

[no] Bruksanvisning

Vannløselig glyserin-gel

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Wateroplosbare glycerinegel

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1 Intended use

Intended purpose

- Prevention of the oxygen-inhibited layer during the polymerization of adhesive luting composites and self-adhesive resin cements
- Try-in of restorations

Patient target group

According to the instructions for use of the selected luting material

Intended users

According to the instructions for use of the selected luting material

Special training

None

Use

For dental use only.

Description

Liquid Strip is an oxygen-impermeable, water-soluble glycerine gel used as accessory during the application of luting materials. It supports complete polymerization of adhesive luting composites and self-adhesive resin cements, as well as the try-in of restorations.

Indications

None

Areas of application:

- Prevention of the oxygen-inhibited layer during the polymerization of adhesive luting composites and self-adhesive resin cements
- Try-in of restorations

Contraindications

The use of this product is contraindicated if the patient is known to be allergic to any of the ingredients.

Limitations of use

Application tip must not be reused

Side effects

When used for the try-in of restorations:

- The hygroscopic material causes fluid movement in the dentin tubules and may therefore trigger temporary sensitivity in the patient. Therefore, the try-in should, if necessary, be performed after desensitization with a desensitizer or under anaesthesia.

Interactions

When used for the try-in of restorations:

Before final placement, thoroughly rinse the material off the restoration and the tooth with water. Residues may impair the adhesion of the luting material.

Composition

Glycerine, water, silicon dioxide, aluminium oxide

2 Usage

2.1 Preventing the oxygen inhibition layer

Preparing the product

1. Remove the original cap from the syringe.
2. Attach the application tip to the syringe.

Applying the material

1. Apply the material evenly along the margins of the restoration.

Polymerizing the material

- Polymerize light- and dual-cure luting materials through the glycerine gel layer.
- For self-cure luting materials, leave the glycerine gel in place until the end of the curing time specified by the manufacturer of the luting material.

Removing the material

1. Thoroughly rinse off Liquid Strip using water.

After application

1. Remove and dispose of the application tip.
2. Re-seal the syringe with the original cap.

2.2 Trying-in restorations

Preparing the product

1. Remove the original cap from the syringe.
2. Attach the application tip to the syringe.

Trying-in restoration

1. Apply a small amount of the material to the restoration.
2. Seat the restoration gently, without applying pressure.

NOTICE! Risk of fracture

- **If the restoration cannot be seated without difficulty, check the proximal area and internal surface for interferences.**
- **For brittle ceramic restorations, check the occlusion carefully or only after final cementation.**

Cleaning after the try-in

1. Thoroughly clean and dry the restoration.
The water-soluble material can be easily removed under running water or in an ultrasonic bath.
2. Thoroughly clean the tooth structure with water spray.

After application

1. Remove and dispose of the application tip.
2. Re-seal the syringe with the original cap.

3 Safety information

- The product has been developed solely for use in dentistry. Process according to the Instructions for Use.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, www.ivoclar.com and your responsible competent authority.
- Check to make sure that the packaging and the product are intact and undamaged before use. If in doubt, contact Ivoclar Vivadent AG or your local sales partner.
- Observe the Safety Data Sheet (SDS).

Supporting documents

Document	To be found at:
Current version of Instructions for Use	www.ivoclar.com/eIFU
Structure of Instructions for Use and Warnings	www.ivoclar.com/eIFU
Explanation of symbols	www.ivoclar.com/eIFU
Safety Data Sheet (SDS)	www.ivoclar.com

Disposal information

Contaminated products and remaining stocks must be disposed of according to the corresponding national legal requirements.

4 Shelf life and storage

- Storage temperature: 2–28 °C (36–82 °F)
- Expiry date: see note on syringes and packaging.

5 Legal information

- Keep material out of the reach of children.
- Prior to use, the product must be inspected under the user's responsibility for integrity and suitability.
- All procedures involving the patient must exclusively be performed by qualified dental professionals.
- The manufacturer assumes no liability for any damage caused by use in restoration types not intended by the manufacturer.
- Any liability and warranty claims become void in the event of damage resulting from improper use or use not in accordance with the intended purpose.

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