Variolink® Esthetic LC

[en] Instructions for Use

Polymer-based dental luting material (intraoral light-curing)

Rx ONLY

C € 0123

Manufacturer:
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com

Date information prepared:
2025-09-11 / Rev. 1
web

ivoclar

English

1 Intended use

Intended purpose

Adhesive cementation of indirect, translucent anterior and posterior single-tooth restorations

Patient target group

Patients with permanent teeth

Intended users / Special training

- Dentists
- No special training required.

Use

For dental use only.

Description

Variolink® Esthetic LC is a colour-stable, adhesive luting composite for the permanent cementation of dental restorations. The special filler composition gives Variolink Esthetic LC a very high radiopacity (300% Al). The purely light-curing Variolink Esthetic LC is suitable for

- the permanent cementation of glass-ceramic, lithium disilicate glass-ceramic and composite resin restorations (inlays, onlays and veneers) with a low thickness of <2 mm and sufficient translucency (e.g. IPS e.max® HT).

The following overview provides recommendations for the selection of the most suitable Variolink Esthetic LC shade:

Shade	Effect	Area of use	
		Veneers	Inlays / onlays
Light +	considerably lightens	✓	—*
Light	slightly lightens	✓	✓
Neutral	no shade effect	✓	✓
Warm	slightly darkens / more yellowish	✓	✓
Warm +	considerably darkens / more yellowish	✓	—*
White opaque	masks discoloured tooth structure	✓	—*

* The use of the shades Light+, Warm+ and White opaque may lead to visible restoration margins when cementing inlays/onlays.

Technical data

Classification acc. to ISO 4049

- Type 2 / Class 2 / Group 1
- Light-curing luting composite that is light-activated intraorally
- Variolink Esthetic LC cures with light in the wavelength range of 400–500 nm.

Shades

Variolink Esthetic LC is available in the following six shade gradations:

Shade		Effekt
Light +	Translucency approx. 5% (opaque)	Lighter / more whitish
Light	Translucency approx. 10%	✓
Neutral	Translucency ≥ 17%	None
Warm	Translucency approx. 12%	✓
Warm +	Translucency approx. 8.5% (opaque)	Darker / more yellowish
White opaque	Translucency approx. 2.5% (opaque)	Masking of discoloured tooth structure

The gradation of Variolink Esthetic LC shades is based on the effect a cement exerts on the brightness value of the final restoration. Variolink Esthetic LC Neutral does not affect the brightness value. At the same time, it demonstrates the highest translucency and is thus neutral in shade. Light and Light + make the restorations lighter, while Warm and Warm + create a darker overall shade impression. The White opaque shade is suitable for masking discoloured tooth structure. In order to evaluate the overall effect of the restoration in conjunction with the various Variolink Esthetic LC shades prior to permanent cementation, we recommend using the Variolink Esthetic Try-In Pastes.

Indications

Missing tooth structure in anterior and posterior teeth

Areas of application

- Permanent cementation of glass-ceramic, lithium disilicate glass-ceramic and composite resin restorations (inlays, onlays and veneers) with a low thickness of <2 mm and sufficient translucency

Contraindications

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- Not suitable for the cementation of opaque restorations.
- Only for translucent restorations.
- Do not use if a dry working field cannot be established or the prescribed application technique cannot be applied.
- The product is not intended to be reprocessed or reused.

Side effects

Systemic side effects are not known to date. In individual cases allergic reactions to individual components have been reported.

Interactions

- Phenolic substances (e.g. eugenol, wintergreen oil) inhibit polymerization. Consequently, the application of products containing these components, e.g. mouth rinses and temporary cements, must be avoided.
- Disinfectants with an oxidative effect (e.g. hydrogen peroxide) may interact with the initiator system, which in turn may impair the curing process. Therefore, do not disinfect the preparation using oxidative agents.
- Alkaline jet media (e.g. Airflow) on dentin can compromise the effect of self-etching adhesives.
- Hemostatic substances may inhibit polymerization and/or lead to discolouration. Therefore, the directions for use of these substances must be observed.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

Si-Zr mixed oxide, ytterbium trifluoride, UDMA, aromatic methacrylate, GDMA, aromatic-aliphatic UDMA, D3MA

The total content of inorganic filler is approx. 40 vol%.

Particle size of inorganic fillers: between 0.15 µm and 15.5 µm.

2 Application

For more detailed information also refer to the instructions for use of the products used in conjunction with Variolink Esthetic LC.

2.1 Removal of the temporary restoration and cleaning of the cavity

Remove any possible residue of the temporary luting cement from the cavity or the preparation with a polishing brush and an oil- and fluoride-free cleaning paste (e.g. Proxylt® fluoride-free prophyl paste). Rinse with water spray. Subsequently, dry with oil- and moisture-free air. Avoid overdrying.

Note: Cleaning with alcohol can lead to dehydration of dentin.

2.2 Try-in of the restoration and isolation

Seat the restoration using the desired Variolink Esthetic Try-In Paste and check the shade, fit and occlusion of the restoration. Care should be taken when checking the occlusion of fragile and brittle ceramic objects before they are permanently cemented, as there is a risk of fracture. If necessary, make adjustments with fine diamonds at medium speed and with light pressure and adequate water cooling. Polish ground surfaces.

Adequate relative or absolute isolation using suitable auxiliaries, such as OptraGate® or OptraDam Plus, is required. If the dental hard tissue is contaminated with blood or saliva during the try-in procedure, it needs to be cleaned again as described in section 2.1.

2.3 Pre-treatment of the restoration

2.3.1 If a restoration which has been pre-treated in the dental laboratory is tried in, the bonding surface of the restoration needs to be prepared as follows after the try-in:

- Following try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.
- Shake Ivoclean before use and completely coat the bonding surface of the restoration with the product using a microbrush or brush.
- Leave Ivoclean to react for 20 s, then thoroughly rinse with water spray and dry with oil-free air.
- Next, prime the bonding surface of the restoration with a suitable bonding agent (e.g. Monobond® Plus). Make sure to observe the instructions for use of the bonding agent used.

2.3.2 Restorations that have not been pre-treated by the dental laboratory should be rinsed with water spray and dried after the try-in procedure.

2.3.2.1 Subsequently, condition the bonding surface of the restoration as follows:

- a) Glass-ceramic restorations (e.g. IPS Empress®)
 - Etch the restoration with 5% hydrofluoric acid (e.g. IPS® Ceramic etching gel) for 60 s or according to the instructions of the manufacturer of the restorative material.
 - Thoroughly rinse the restoration with water spray and dry with oil-free air.
- b) Lithium disilicate glass-ceramic restorations (e.g. IPS e.max® Press, IPS e.max CAD)
 - Etch the restoration with 5% hydrofluoric acid (e.g. IPS Ceramic etching gel) for 20 s or according to the instructions of the manufacturer of the restorative material.
 - Thoroughly rinse the restoration with water spray and dry with oil-free air.

2.3.2.2 Apply Monobond Plus with a brush or a microbrush to the pre-treated surfaces, let it react for 60 s and then disperse with a strong stream of air.

2.3.3 Alternatively, restorations made of glass-ceramics and lithium disilicate glass-ceramics (e.g. IPS Empress, IPS e.max Press, IPS e.max CAD) may be conditioned with Monobond Etch & Prime® as follows:

- Following try-in, thoroughly rinse the restoration with water spray and dry with oil- and moisture-free air.
- Apply Monobond Etch & Prime on the bonding surface using a microbrush and scrub it into the surface for 20 s. Allow to react for another 40 s.
- Thoroughly rinse off Monobond Etch & Prime with water spray until the green colour has disappeared. If any residue remains in areas showing microporosity, the restoration may be cleaned with water in an ultrasonic unit for up to 5 min.
- Dry the restoration with a strong stream of oil- and moisture-free air for approx. 10 s.

2.3.4  Composite and fibre-reinforced composite restorations should always be conditioned according to the instructions of the manufacturer of the restorative material.

With respect to Tetric® CAD restorations, please observe the directions stated in the Tetric CAD Instructions for Use.

2.4 Pre-treatment of the preparation and application of the adhesive (e.g. Adhese® Universal)

 Please observe the instructions for use of the adhesive used.

2.4.1 Conditioning with phosphoric acid gel (optional)

The bond to enamel can be improved by selectively etching the enamel or by applying the "etch & rinse" technique. Unprepared enamel surfaces must be conditioned with phosphoric acid gel (e.g. Total Etch). Please observe the instructions for use of the phosphoric acid gel.

a) Selective enamel etching

Apply phosphoric acid gel (e.g. Total Etch) onto the enamel and allow it to react for 15–30 s. Then rinse thoroughly with a vigorous stream of water for at least 5 s and dry with compressed air until the etched enamel surfaces appear chalky white.

b) Etch & rinse technique

Apply phosphoric acid gel (e.g. Total Etch) onto the prepared enamel first, and then to the dentin. The etchant should be left to react on the enamel for 15–30 s and on the dentin for 10–15 s. Then rinse thoroughly with a vigorous stream of water for at least 5 s and dry with compressed air until the etched enamel surfaces appear chalky white.

2.4.2 Application of Adhese Universal

- Starting with the enamel, thoroughly coat the tooth surfaces to be treated with Adhese Universal.
- The adhesive must be scrubbed into the tooth surface for at least 20 s. This time must not be shortened. Applying the adhesive on the tooth surface without scrubbing is inadequate.
- Disperse Adhese Universal with oil- and moisture-free compressed air until a glossy, immobile film layer results.
Important note: Avoid pooling, as this may compromise the accuracy of fit of the final restoration.
- Light-cure Adhese Universal for 10 s using a light intensity of $\geq 500 \text{ mW/cm}^2$ (e.g. Bluephase® G4).

2.5 Seating of the restoration

- Variolink Esthetic LC is sensitive to blue light (operatory light, ambient light). Therefore, Variolink Esthetic LC should not be dispensed from the syringe until right before use. Exposure to intensive light should be avoided during application.
- Apply Variolink Esthetic LC with the application tip or a brush/spatula directly to the internal surface of the restoration.

Note: Direct application of Variolink Esthetic LC to a preparation that has been pre-treated with adhesive may – depending on the adhesive used – lead to an acceleration of the curing process and affect the accuracy of fit of the restoration.

- Seat the restoration and **fix/hold it in place during excess removal.**

- Remove all excess luting material.

a) Wiping technique

Remove excess material immediately with a microbrush/brush/foam pellet/dental floss or scaler. Make sure to remove excess material in time, especially in areas that are difficult to reach (proximal and gingival margins).

- b) Circular technique suitable for veneers/inlays/onlays – Light-curing of excess and subsequent removal
Light-cure excess material with a polymerization light (light intensity $\geq 500\text{mW/cm}^2$; e.g. Bluephase G4) for **2 s** at a distance of 10–15 mm by running the light probe along the entire cement line.

In the case of **laminare veneers**, choose a start and end point in the incisal area and light-cure the cement line by moving the polymerization light in a circle in a clockwise direction.

In the case of **inlays/onlays**, choose a start and end point in the mesial or distal region and light-cure the cement line by performing a circular movement with the polymerization light.

Thereafter, excess cement is easy to remove with a scaler. Make sure to remove excess material in time, especially in areas that are difficult to reach (proximal areas, gingival margins, pontics).

2.6 Polymerization

- As with all composite systems, Variolink Esthetic LC is subject to oxygen inhibition. In other words, the surface layer (approx. 50 μm) does not polymerize during the curing process, as it comes in contact with atmospheric oxygen. To prevent this, cover the restoration margins with glycerine gel/Airblock (e.g. Liquid Strip) immediately after excess removal.
- Polymerize Variolink Esthetic LC in segments, starting with the proximal margins:

Light intensity	Curing time per mm ceramic and segment
500 – 800 mW/cm^2	20 s
900 – 1,300 mW/cm^2	10 s

- Rinse off glycerine gel/Airblock (e.g. Liquid Strip).

2.7 Finishing of the completed restoration

- Check occlusion and functional movements and make adjustments if necessary.
- Finish the cement lines with finishing diamonds if necessary.
- Smooth out the cement lines using finishing and polishing strips and polish them with suitable polishing instruments (e.g. OptraGloss®).
- If necessary, finish the restoration margins with suitable polishers (e.g. OptraGloss).

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclar.com).
- Explanation of symbols: www.ivoclar.com/eIFU
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
- Basic UDI-DI: 76152082ACEME004E3

Warnings

- Observe the Safety Data Sheet (SDS) (available in the download section of the Ivoclar Vivadent AG website www.ivoclar.com).
- Prevent any contact of uncured Variolink Esthetic LC with the skin/mucous membrane and eyes.
- Uncured Variolink Esthetic LC may cause slight irritation and may lead to a sensitization against methacrylates.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks. Some of these risks are listed below:

- Failure of the adhesive bond

4 Shelf life and storage

- Storage temperature 2–28 °C / 36–82 °F
- Close Variolink Esthetic LC syringes immediately after use. Exposure to light causes premature polymerization.
- Do not use the product after the indicated date of expiration.
- Date of expiration: see information on syringes and packages.

Before use, visually inspect the packaging and the product for damage. In case of any doubt, consult Ivoclar Vivadent AG or your local trade partner.

5 Additional information

Keep material out of the reach of children!

Not all products are available in all countries.

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions.

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