Variolink® Esthetic Try-In Paste

[en] Instructions for Use

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1 Intended use

Intended purpose

Intraoral shade determination for indirect restorations before permanent cementation with the corresponding luting material

Patient target group

According to the instructions for use of the selected luting material

Intended users

According to the instructions for use of the selected luting material

Special training

None

Use

For dental use only.

Description

Variolink® Esthetic Try-In Pastes are water-soluble glycerine pastes used as accessory during the application of the luting material Variolink® Esthetic LC/DC, Variolink® 100.

The try-in pastes exhibit the same shade and translucency as the cured luting material when applied in the same layer thickness. This allows for a realistic simulation of the shade effect of indirect restorations.

Shades

The try-in paste is available in the following shades:

Shade	Effect
Light+	Lighter/more whitish
Light	✓
Neutral	None
Warm	✓
Warm+	Darker/more yellowish
White opaque	Masking of discoloured tooth structure

The try-in pastes differ slightly in shade from the uncured luting materials, as the latter achieve their final colour only after polymerization.

Indications

None

Areas of application:

Intraoral shade determination for indirect restorations before permanent cementation with the corresponding luting material

Contraindications

The use of this product is contraindicated if the patient is known to be allergic to any of the ingredients.

Limitations of use

Use only with Variolink® Esthetic LC/DC, Variolink® 100.

Side effects

- The hygroscopic material causes fluid movement in the dentin tubules and may therefore trigger temporary sensitivity in the patient. Therefore, the try-in should, if necessary, be performed after desensitization with a desensitizer or under anaesthesia.
- In rare cases, components of the product may lead to sensitization

Interactions

Before final placement, thoroughly rinse the material off the restoration and the tooth with water. Residues may impair the adhesion of the luting material.

Composition

Glycerine, water, silicon dioxide, aluminium oxide, dimethacrylate, inorganic pigments

2 Usage

Preparing the product

1. Select the try-in paste in the desired shade of the luting material.
2. Remove the original cap from the syringe.

Trying-in restoration

Note: Simulation of the shade effect should be performed before isolating or drying the tooth structure, as drying temporarily lightens the tooth structure.

1. Apply a small amount of the material to the restoration.
2. Seat the restoration gently, without applying pressure.

NOTICE! Risk of fracture

- **If the restoration cannot be seated without difficulty, check the proximal area and internal surface for interferences.**
- **For brittle ceramic restorations, check the occlusion carefully or only after final cementation.**

Cleaning after the try-in

1. Thoroughly clean and dry the restoration.
The water-soluble material can be easily removed under running water or in an ultrasonic bath.
2. Thoroughly clean the tooth structure with water spray.

After application

1. Re-seal the syringe with the original cap.

3 Safety information

- The product has been developed solely for use in dentistry. Process according to the Instructions for Use.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, www.ivoclar.com and your responsible competent authority.
- Check to make sure that the packaging and the product are intact and undamaged before use. If in doubt, contact Ivoclar Vivadent AG or your local sales partner.
- Observe the Safety Data Sheet (SDS).

Warnings

CAUTION! Product contains methacrylates. Irritation of eyes, respiratory tract and skin as well as sensitization are possible. Contact with the skin/mucous membrane or eyes should be avoided. Conventional medical gloves do not provide adequate protection.

Supporting documents

Document	To be found at:
Current version of Instructions for Use	www.ivoclar.com/eIFU
Structure of Instructions for Use and Warnings	www.ivoclar.com/eIFU
Explanation of symbols	www.ivoclar.com/eIFU
Safety Data Sheet (SDS)	www.ivoclar.com

Disposal information

Contaminated products and remaining stocks must be disposed of according to the corresponding national legal requirements.

4 Shelf life and storage

- Storage temperature: <30 °C (86 °F)
- Expiry date: see note on syringes and packaging.

5 Legal information

- Keep material out of the reach of children.
- Prior to use, the product must be inspected under the user's responsibility for integrity and suitability.
- All procedures involving the patient must exclusively be performed by qualified dental professionals.
- The manufacturer assumes no liability for any damage caused by use in restoration types not intended by the manufacturer.
- Any liability and warranty claims become void in the event of damage resulting from improper use or use not in accordance with the intended purpose.

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